


SIX (6) STEPS FOR CHAIRSIDE PROCESSING OF LOCATOR ATTACHMENTS INTO OVERDENTURES / BITE BLOCKS

1

PLACE a White Block Out Spacer Ring around each abutment and press it down to the tissue. Place a Denture Cap with a Black Processing Male onto each abutment and press down firmly to ensure engagement of the abutment.

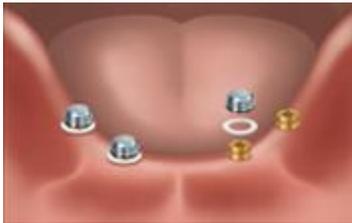

2

APPLY fit check marking paste inside of the denture. Insert it into the mouth in position over the Denture Caps. Mark the areas where the denture will need to be relieved to allow space for the caps to be picked up.




3

RELIEVE the areas marked with an acrylic bur. Try in the denture to verify that the Denture Caps are not in contact with the acrylic in any area. Cut lingual/palatal vent windows in the denture to visualize full seating and for excess material to vent. Cut an undercut below the intaglio surface of each Denture Cap relief area for mechanical retention.



4

DRY the Denture Caps. Apply a small amount of CHAIRSIDE Material around the circumference of each cap. Place CHAIRSIDE Material into the relief areas of the denture and seat it over the caps and onto the tissue. Have the patient close into light occlusion and hold while the CHAIRSIDE Material sets. The set time of the material from the time the material is mixed to the time it is set is five (5) minutes. The working time for the material is two (2) minutes, fifteen (15) seconds.

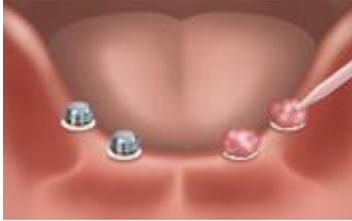

5

DISENGAGE the denture from the abutments and remove from the mouth. Verify that the Denture Caps have been securely processed into the denture. Fill any voids, light cure and polish the denture. The material will bond to itself and will cure within 30 seconds with light application.



6

PLACE the denture in the mouth and press down to engage the males on the attachment system.



Questions?

Contact Gardali Advanced Restorative Solutions at: 800.773.6767